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USER MANUAL

SYMBOLS	11
INTRODUCTION	11
INSTALLATION	11
SPECIFICATIONS.....	12
PRODUCT PROFILE	13
PACKING LIST.....	14
INSTALLATION.....	15
WARNING - OPERATING INSTRUCTIONS	17
MAINTENANCE	17
ACCESSORIES.....	19
WARRANTY	19

SYMBOLS

When paragraphs are marked by this symbol, they contain instructions that must be carried out carefully in order to prevent damage to the device and to the operator

INTRODUCTION

Thank you for choosing our demineraliser from C.B.M. s.r.l.. Before carrying out any operations we recommend to read carefully this instruction manual and to file it for future consultations. The current manual strives to give the user the instructions for a correct, efficient and safe functioning. The device must be used in compliance with the procedures contained in the manual. It must never be used for other aims. The user is legally responsible for the installation and the functioning of the device. C.B.M. s.r.l. reserves the right to improve or modify the current manual or even the device without notice.

INSTALLATION

We recommend to follow all the procedures of installation, use and maintenance, included in the manual. The company is responsible for the safety and it guarantees the product only if maintenance, repairs and modifications will be made exclusively by authorized technicians or by the manufacturer himself. We heartily recommend to use our spare parts in order to have a long-lived and working product.

Always disconnect the water supplies at the end of the day.
C.B.M. s.r.l. cannot be held liable for damage caused by failure to perform these operations.

SPECIFICATIONS

Demineralizer - Codice: SST-DEM

Flow rate	48 l/h
Water purity	0.1 μ S
Maximum water temperature	30°C
Internal dimensions of cartridge (d x h)	72,5 x 250 mm
Overall dimensions (w x h x l)	270 x 335 x 145 mm
Net weight	2,5 Kg
Assembly time	30 minuti

Resin : Code: SST-DEMBID

Chemical structure	Polistirene reticolato con DVB
Type	Letto misto pronto all'uso
Physical Form and Appearance	Sfere gialle traslucide
Functional Group	Solfonico, trimetilamminico
Ionic form, as shipped	H+ / H-
Exchange capacity	Catione Minimo 2,0 eq/l - anione Minimo 1,3 eq/l
Volume ratio	40% cationica / 60% anionica
Humidity	55/60%
Particle Size	0,3 / 1,18 mm
Shipping weight	705 / 740 g/l
Conductivity	1 colonna < 5mS/cm // 2 colonne < 0,5mS/cm
pH range	6,5 - 7,5
Packing	Barattolo 1,2 litri

PRODUCT PROFILE

Universal demineralisers from C.B.M. s.r.l. eliminate dissolved salts from the water supply. Columns containing ion exchange resin absorb dissolved mineral salts and deliver pure, demineralised water. Depending on the quantity of salts dissolved in the water supply, demineralised water leaving the unit has a quality of 0.1 μS to 30 μS . Delivered water conforms to the requirements of EN 13060 governing the purity of water supplies to small steam sterilisers and other equipment.

- | | |
|--|-------------------------------------|
| A. Fixing plate | D – Tube 6,35 x 4,35 |
| B – Flow regulator | E – Container of resin (1,2 litres) |
| C – Filter cartridge holders + empty filter cartridges | F - Wrench |

Resin

The universal, mixed bed resin is of excellent quality and able to demineralise water to a value very near 0 μS . The resin must be replaced at intervals depending on the conductivity of the water supply. For example, if the water supply has a conductivity of 280 μS , the resin must be replaced approximately after ever 80 litres of pure water. After this time, water purity will be poorer than 30 μS and will no longer conform to the requirements of EN 13060 standard.

The resin can be replaced quickly and easily. You can replace either the entire filter cartridge or just the resin. 1.2 litres of resin is sufficient to regen rate 2 cartridges.

PACKING LIST

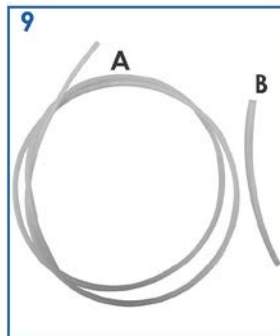
	2 filter cartridge holders
	Container of resin (1.2 litres) – cod: SST-DEMBID
	2 empty filter cartridges – cod: SST-DEMCART
	1 wrench
	1 flow regulator
	1 npt nipples connector $\frac{1}{4}$ for water, 38mm length
	2 npt curved connectors $\frac{1}{4}$ for $\frac{1}{4}$ water tube
	Water tube 6,35x4,35 n. 2 - 1,80 mt length n. 1 - 0,20 mt length
	Fixing plate
	Kit composed from 8 screws and 8 washer

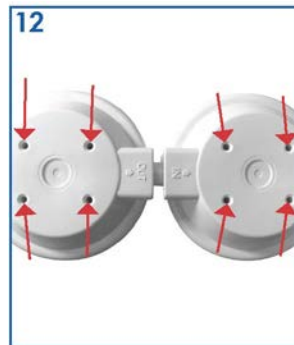
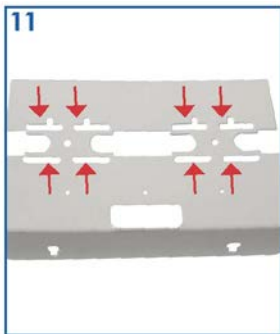
INSTALLATION

Attention! Before to start the assembly, make sure to have the teflon film.

If you want, it is possible to fix the demineralizer to wall (fixing screws not supplied).

WARNING - OPERATING INSTRUCTIONS

Attention: in case it is the first installation you have to dredge the circuit for 40 seconds. The correct value is about 8-12 μ s, if it is clearly high it means there has been a wrong installation.

Attention: values will not be truthful if the measurement happens before the pouring out. In order to obtain a truthful value it is necessary to make water pour out about 10 seconds into the drain of the sink.

The water quality is acceptable only if the measured value is lower than 60 μ s.

If it is higher you have to substitute the resin cartridge

Attention! We recommend to disconnect the water supply when the demineralizer is not working and at the end of the day.
C.B.M. s.r.l.srl cannot be held liable for damage caused by failure to perform these operations.

MAINTENANCE

Before performing any maintenance, disconnect the water supply.
For any problem, contact always the Authorized Technician C.B.M. S.r.l.

ATTENTION: Before removing the cartridges, water pressure should be released, in order to avoid water splashes. Procedure is the following:

- Access to UTILITY for manual water loading of the sterilizer
- Press START to simulate water loading and wait a few seconds,
- Press STOP and exit the program
- Remove cartridges

REPLACEMENT OF THE RESIN

Choose one of the two options:

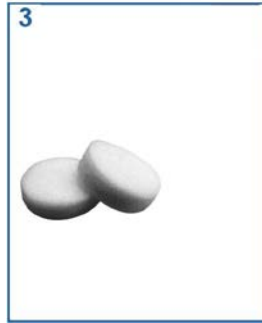
A. Replacement of the whole cartridge composed from resin: code: SST-DEMCART

Repeat the operation for the second cartridge.

B. Replacement only the resin: item DRR12

Repeat the operation for the second cartridge.

ACCESSORIES

IMAGE	CODE	DESCRIPTION
	SST-DEMCART	93/4 cartridge with resin. The universal, mixed bed resin is of excellent quality and able to demineralise water to a value very near 0 µS.
	SST-DEMBID	Container of resin (1.2 litres) The universal, mixed bed resin is of excellent quality and able to demineralise water to a value very near 0 µS.

WARRANTY

C.B.M. guarantees its demineraliser for 2 (two) years against manufacturing or component faults assessed by C.B.M. as not attributable improper use by the operators.
The warranty does not cover wear and tear parts nor repairs or replacements carried out by unauthorized personnel.

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Always disconnect the water supplies at the end of the day.

C.B.M. s.r.l. cannot be held liable for damage caused by failure to perform these operations

During the warranty period the potential replacement will be decided by C.B.M. only if it should be impossible to carry out a repair which will guarantee the correct operation.

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Data, descriptions and illustrations are purely indicative.

C.B.M. S.r.l. reserves the right to make improvement changes to its products without notice.